


The UbiDuo device is used to accommodate people who are deaf, hard of hearing and those who have difficulty communicating with speech or cannot speak at all.

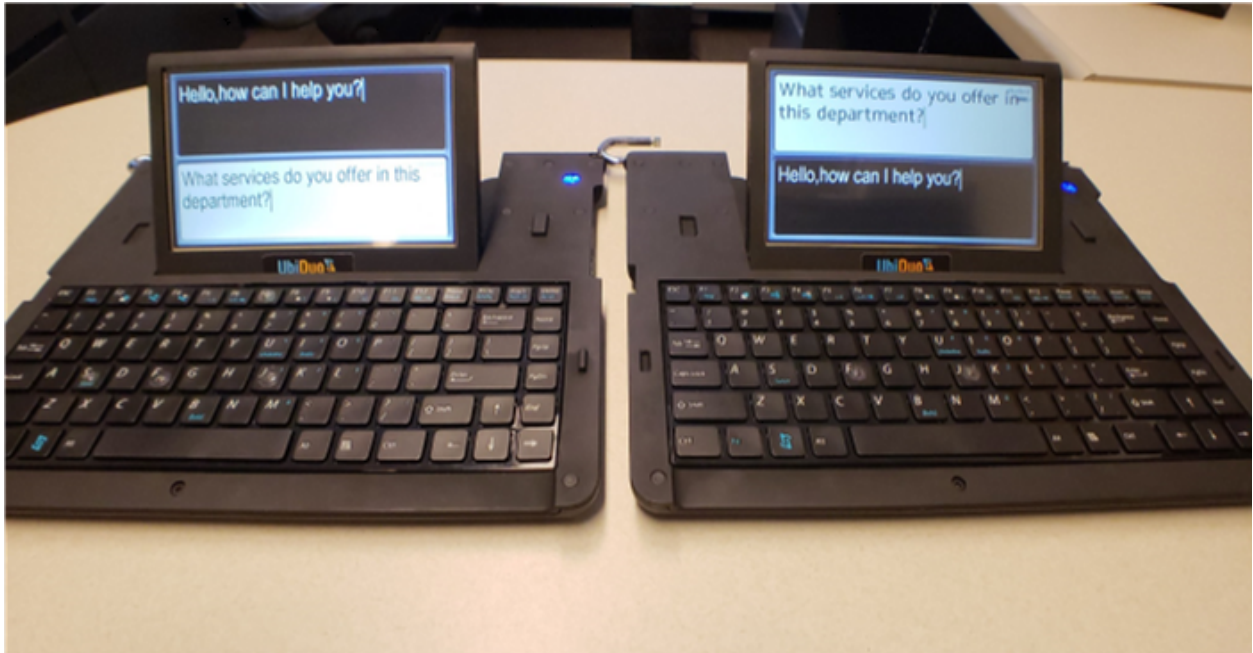

Power The Unit On

Press and hold the power button (a blue plastic button located on the upper right hand corner of each unit) for 2-3 seconds until the blue power indicator light comes on.

Note: Both halves of the UbiDuo must be turned on for a split screen chat to be initiated. The two halves will automatically connect with each other when the split screen chat is selected on one of the halves and the devices are within 300 feet of each other.

Power The Unit Off

Press and release the power button. If an existing conversation has not been explicitly saved (optional), powering the UbiDuo off will erase the conversation from memory.

Typing Text

On both devices the text you type will appear on one half of the screen. The text the student types will appear on the other half of the screen.

Keyboard Shortcuts

Shift + F7.....Clear All Screens—will clear screens on both devices
Ctrl + S.....Save Chat
ESC Key.....Places the device at the main menu
Shift + F1.....Help Menu